



*Audience: Ophthalmology Technicians*

Guide through the **Zeiss Forum Link Net** workflow.

## Tomey EM 3000 Specular Microscopy Instrument

1. Enter the patient ID.
2. Take the measurement.
3. Run your analysis.
4. Hit **Export**.
5. Hit **Export** again.
6. The screen will say **now transmitting**. When done, **Link Net** will automatically appear.

### Link Net Process

1. Use **Today's Worklist** tab on the bottom of the **Link Net** screen.
2. Double click on your patient.
3. Pick which eye.
4. Sends automatically to **Forum**.

## iTrace Instrument

You will have to two options to import your images through **Link Net**.

### Option one:

1. Look up your patient's name or enter it manually.
2. Acquire your image.
3. Select your applicable topography map.
4. Use floppy to send screenshot (what you are seeing on the screen).
5. Save as box appears (C:\ToForum is default path).
6. Click **Save**.
7. Acknowledge message box.

### Link Net steps:

1. **Link Net** will automatically appear.
2. Use **Today's Worklist tab** on the bottom of the **Link Net** screen.
3. Double click on your patient.
4. Pick which eye.
5. Sends automatically to **Forum**.



## Option two:

1. Click **printer** icon.
2. A report will be sent.
3. Click **To Forum** in printer selection.
4. Hit **Print**.

## Link Net steps:

1. **Link Net** will automatically appear.
2. Use **Today's Worklist** tab on the bottom of the **Link Net** screen.
3. Double click on your patient.
4. Pick which eye.
5. Sends automatically to **Forum**.

## Downtime Process for Tomey and iTrace Instruments

- If the **Forum** system is down, print to printer.
- If Epic is down:
  - Create/register patients in **Forum**.
  - Place **Orders** in **Forum**.
- Refer to the tip sheet the Zeiss training team gave you for creating patients and placing orders within **Zeiss Forum Viewer**.