


# CIRRUS to FORUM Data Transfer Quick Reference Guide

Version 5.21 and higher


1. Select **Refresh** to update the “View Today’s Patients” list. This displays both the current work list as well as patients already tested. To view the “Work list” select **Records, Search Work List** option.

Figure 1.

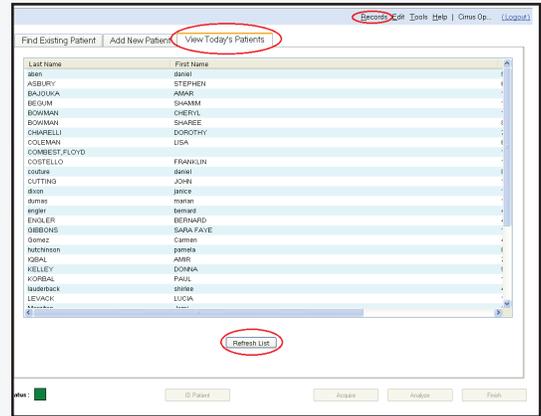

Work List showing untested patients only.

Figure 2.

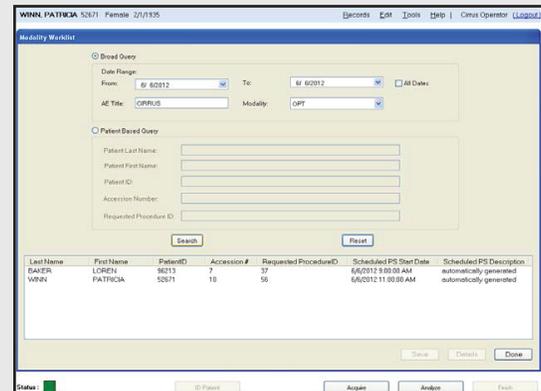

2. Select patient from the worklist, choose **Save** and then **Done**. Locate/select desired patient from the “View Today’s Patient” list. Perform desired testing as usual.

3. To send to FORUM® and/or Print select **Analyze**, select **Eye**, choose **Analysis Protocol**. Click on/hover over Printer icon with mouse pointer. Choose **Export to DICOM** to send to FORUM and/or **Print No Preview** to send report to printer.

If instrument configured in FORUM/DICOM mode, scan(s) will be automatically archived after selecting **Finish**.

Figure 3.

