# HFA to FORUM Data Transfer Quick Reference Guide

# **New Tests**

- **1.** Select the test type and right or left eye.
- 2. From the Patient Data screen, select Recall Patient Data.

Figure 1. Patient Data Screen

	08-20-2009	3:22 PM
li	Patient Data 1	
Patient ID	Clear Pa	tient Data
Patient Name	Recall Pa	atient Data

### Figure 2. Source Drop Down Menu

	Source Options
Source:	
Work List	Ŧ

# Figure 3. Worklist

		09-25-2009	11:13 AM	
i	Work L	ist		
Patient Name	Patient ID	Accession		
Forest, Green	887766	27		
McBrian, Thomas	CZM081222004	С2м090128001		
				1.0
Requested Procedure Procedure Description	ID: :			
Procedure Start Date:				
Cancel	Detail		Proceed	

# Figure 4. Test Completion Display

Do You Wish To Save and Transmit This Test For:		
Zeiss Carl Joseph Dr.		
DOB: 09-22-1943		
Yes Change Patient Data First No		

4. Select the patient from Work List and click **Proceed**.

**3.** Select **Worklist** from the Source drop down menu.

**5.** Perform the visual field test.

- 6. When complete, answer Yes.
- 7. The test is sent to FORUM<sup>®</sup>.

If testing both eyes, complete testing to both before printing.

When done, select Print icon.

Select desired print protocols, if necessary, then choose Print All Selected Items.

**To Send/Reprint Previous Tests** 

Enter patients name, click Proceed.

Change source from hard drive to DICOM Archive.

Select all or desired exams to send, select Proceed.

Select **Print** icon.

If doing GPA or Change Analysis, use **Exam Selection** button to set baseline/follow-ups.

#### Figure 5. Print Selection



#### Figure 6.

i	Dicom Archive	07-11-2007	5:01 PM	
Full Threshold exams are allowed as follow-up when the baseline is Full Threshold.				
Name	DOB Eye Test	Date		
>GPA_TC_1	01-01-1931 R SS-30-2	11-20-2002		
>GPA_TC_1	01-01-1931 R SS-30-2 I	04-18-2002		
*GPA_TC_1	01-01-1931 R SS-30-2 I	04-18-2001		
*GPA_TC_1	01-01-1931 R SS-30-2	12-15-1998		$\left  \mathcal{D} \right $
GPA_TC_1	01-01-1931 R SS-30-2	11-27-1997	Ŧ	
			I	
> Follow-up * Baseline	2 ! Excessive False SITA-Standard Positives	0		
Selection Mode	Baseline Select Defaul	Pro	oceed	Ŭ



Carl Zeiss Meditec, Inc. 5160 Hacienda Drive Dublin, CA 94568 USA www.meditec.zeiss.com



EC REP Carl Zeiss Meditec AG Goeschwitzer Str. 51-52 07745 Jena Germany www.meditec.zeiss.com

2660021148936 A



