


IOLMaster 500 to Forum Data Transfer


1. On the IOL from the “Patient list” tab click on “Waiting Room”.




2. The first patient of the day will open the Worklist as shown. Click on the “Broad Query” tab.

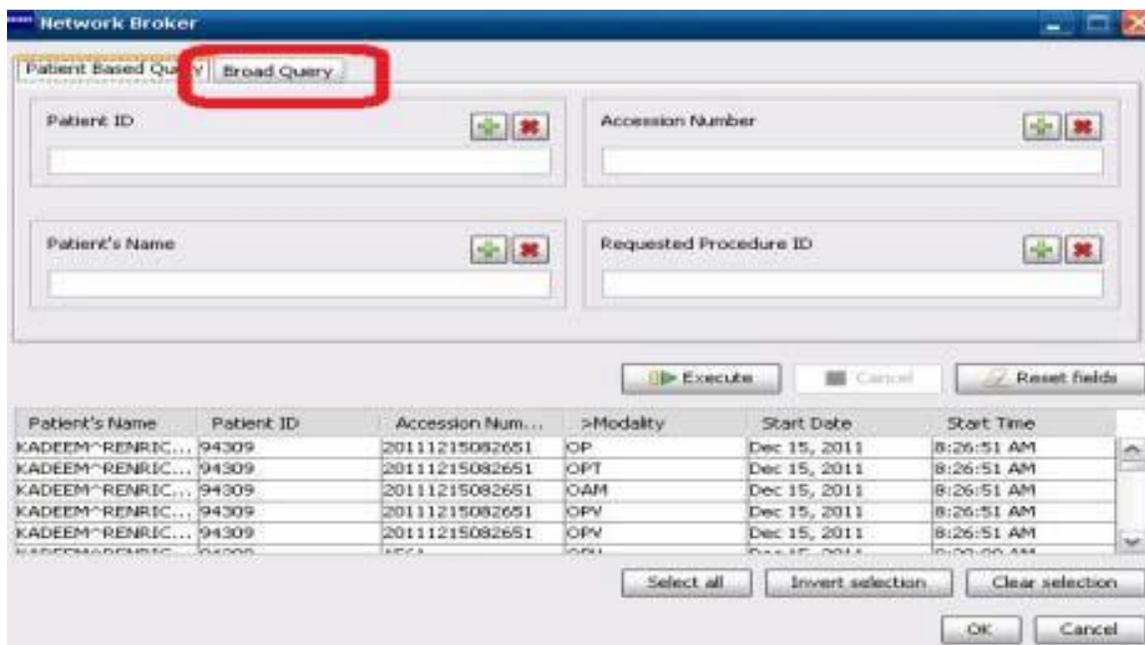

3. Enter today's date and in "Modality" enter OAM click the "Execute" button. IOLMaster will remember the tab selection, OAM and date until the unit is turned off. After the first patient just click "execute".

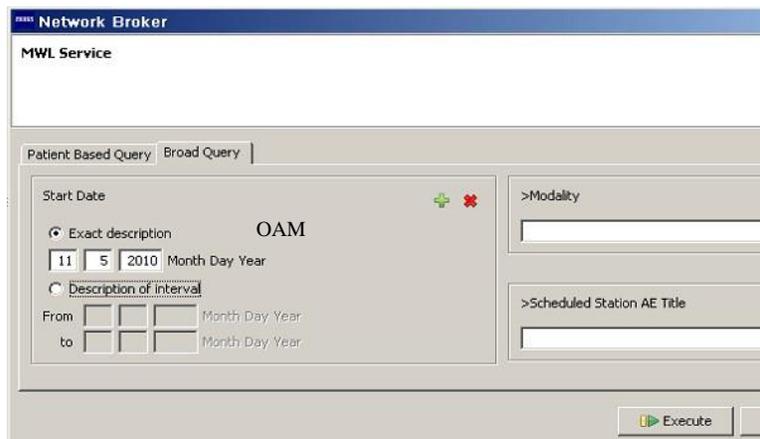

4. Highlight the patient from the list, Click "OK". Proceed with the exam as always. Do all calculations and return to this screen. Click on the computer icon to transmit the report to Forum. **This is a multi page report. Use the center set of arrows in Forum to page through the report.**

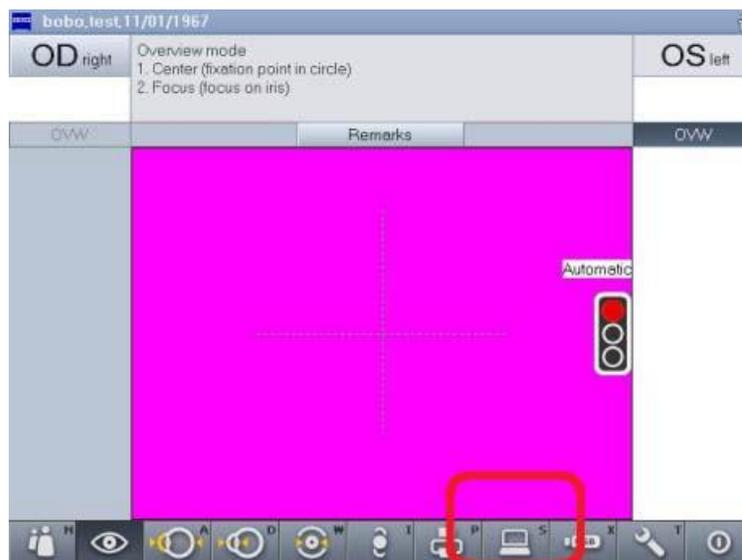

5. For prior tests follow the instructions in Step 4 only.

