


HEYEX™ DICOM Interface - Quick Guide

for HEYEX, Version 1.7 and SPECTRALIS®, Version 5.4

Before Image Acquisition: Select Patient from Worklist




1. Click **"Worklist"**. The "Worklist" window opens.

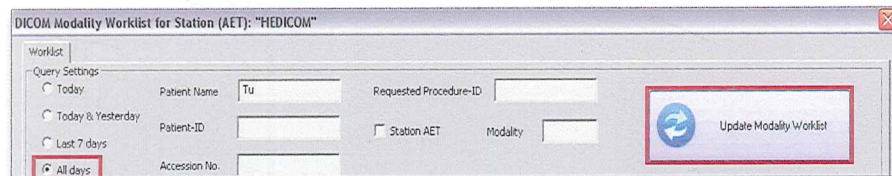


2. Enter query criteria and click **"Update Modality Worklist"**. The query results are displayed.

Date/Time	*Patient Name	Date of Birth	Sex	Patient-ID
3/22/2011 9:12 PM	August^Arnold^Paul^^	8/8/1958	M	808
3/22/2011 9:12 PM	Monday^Montel	1/1/2001	M	1301
3/22/2011 9:12 PM	September^Steve	9/9/1909	M	909
3/22/2011 9:12 PM	Tuesday^Tim	2/2/2002	M	1401

3. **Double-click on an order**. The patient data is loaded into HEYEX and the "Order Detail" window opens.

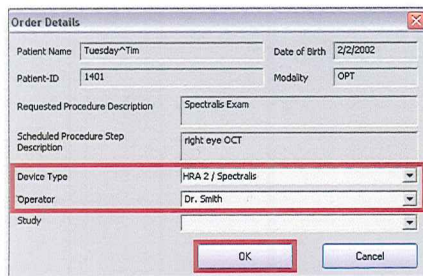


4. **Select device and operator** and click **"OK"**. The "Eye Data" window opens.


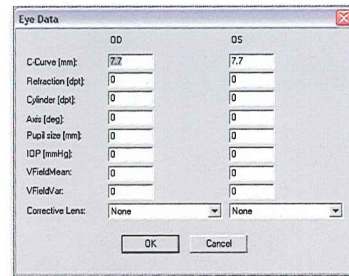


5. **Continue as usual**.

After Image Acquisition: Send Images to the PACS

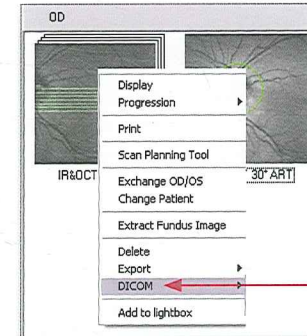

1. **Right-click** on the image(s) you want to send, then select **DICOM**.

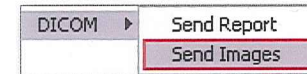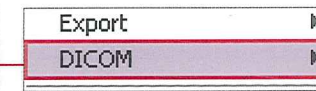


2a. To send images click **"Send Images"**.


or




2b. To send a report click **"Send Report"**. The DICOM Report window opens.

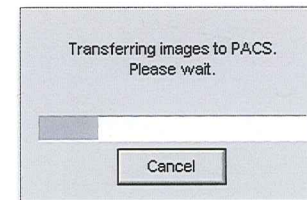

3a. The images will be sent.

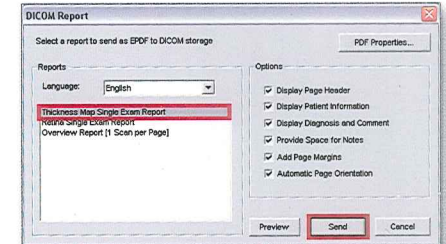

3b. **Select a report** and click **"Send"**.

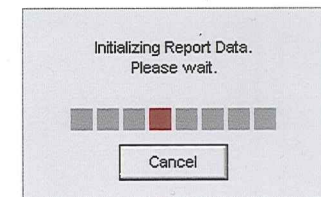

4b. The report will be sent.